

## Life is on

### Unique Roger features

#### Improved speech understanding<sup>1</sup>

The Roger technology continuously measures the surrounding noise level and adapts the microphone's volume automatically. If the noise level increases, Roger adapts accordingly, keeping the speaker's voice above the background noise.

#### Enhancing group listening experience<sup>2</sup>

MultiBeam Technology enables you to experience exceptional speech understanding in group conversations. By utilizing multiple microphones in six directions, speech from 360 degrees is calculated and compared. The direction with the best speech clarity is automatically selected.

#### Access to multiple talkers

Unique to Roger and fully automatic, MultiTalker Network allows several Roger microphones to be used together, providing you with access to multiple talkers in any situation.

### Getting Roger for you

Roger systems may be reimbursed for work. If you are eligible, your hearing care professional can help you to apply.

Find out more about Roger at [www.phonak.com/roger](http://www.phonak.com/roger)

<sup>1</sup> Thibodeau, L. (2014). Comparison of speech recognition with adaptive digital and FM wireless technology by listeners who use hearing aids. American Journal of Audiology, 23(2), 201-210

<sup>2</sup> Based on preliminary data. Peer-reviewed article and Field Study News in preparation, available end of 2019 at [www.phonakpro.com/evidence](http://www.phonakpro.com/evidence).

At Phonak, we believe that hearing well is essential to living life to the fullest. For more than 70 years, we have remained true to our mission by developing pioneering hearing solutions that change people's lives to thrive socially and emotionally. Life is on.

[www.phonak.com](http://www.phonak.com)



## Roger™ wireless solutions at work

**sonova**  
HEAR THE WORLD

028-1822-02 / V2.00 / 2019-09 / cl / Printed in xxx © 2019 Sonova AG All rights reserved

A Sonova brand

**PHONAK**  
life is on

# Bridging the understanding gap

The Roger portfolio of wireless microphones and receivers gives you better speech understanding over distance and in noise. Microphones pick up the speaker's voice and transmit it to receivers connected to, or installed in your hearing aids.

## Roger microphones



### Roger Table Mic II

Designed for meetings; several microphones can be connected to cover large meetings.



### Roger Select™

Lying on a table it automatically selects the person who is talking, it can be worn by a presenter and has streaming for phone calls.



### Roger Pen™

With its handy, portable design it can be placed on a table or worn by a presenter and has streaming for phone calls.

## Roger receivers



We offer different receiver solutions, depending on the type of hearing aids or implants you have. Choose between Phonak hearing aids featuring RogerDirect™, design-integrated receivers, universal mini receiver or neckloop receiver for any hearing aid with a t-coil.

## Roger in different work situations




### Small meetings

Place one microphone on the meeting table and you will hear your colleagues clearly.



### Large meetings

In large meetings multiple Roger Table Mic II will do an excellent job picking up all voices.

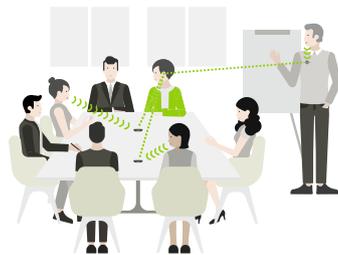


### Meetings with presenters

Connect the presenter's Roger microphone to the table microphone and you can hear speech clearly from all participants.



### One-to-one conversation in noise

In distracting background noise, place a Roger microphone around your colleague's neck or clip it on to the clothing.

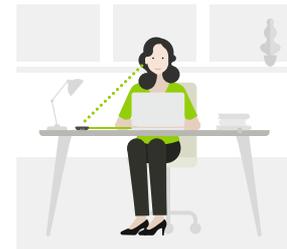

### Multimedia

Connect your Roger microphone, using the audio cable, for video conferences or internet calls.



### Phone calls

By connecting Roger Pen or Roger Select to your phone you can stream phone calls directly to your hearing aids or implants.